

## Cleaning and disinfecting

- 1 Open the shipping carton and place the pump in its plastic bag under the hood.
- 2 Verify that the customer included a Declaration of cleaning and disinfection as required. If cleaning cannot be verified do not continue, notify customer/supervisor.
- 3 Remove pump from plastic and discard bag.
- 4 Check that exhaust port on bottom of unit is free from obstructions, dust or foreign materials.
- 5 Apply germicidal/antibacterial product to a cotton swab and clean the exhaust port.



- 6 Carefully clean all surfaces of the pump, power cord, canister holders and universal holder (if returned) with germicidal / antibacterial treated disposable wipe. Pay close attention to the handle and areas closest to the location of the canister. For stubborn or larger debris use a soft bristle brush.
- 7 Use a second wipe to thoroughly clean the surface.
- 8 Allow to remain wet for 5 minutes
- 9 Dry surfaces with a disposable towel or cloth.

**Do not immerse any part of the pump in fluid or use unnecessarily wet cloth.**

### Wound Management

Smith & Nephew, Inc.      Customer Care Center      [www.smith-nephew.com](http://www.smith-nephew.com)  
970 Lake Carillon Drive      1-800-876-1261      [www.npwt.com](http://www.npwt.com)  
Suite 110      T 727-392-1261  
St. Petersburg, FL 33716      F 727-392-6914  
USA